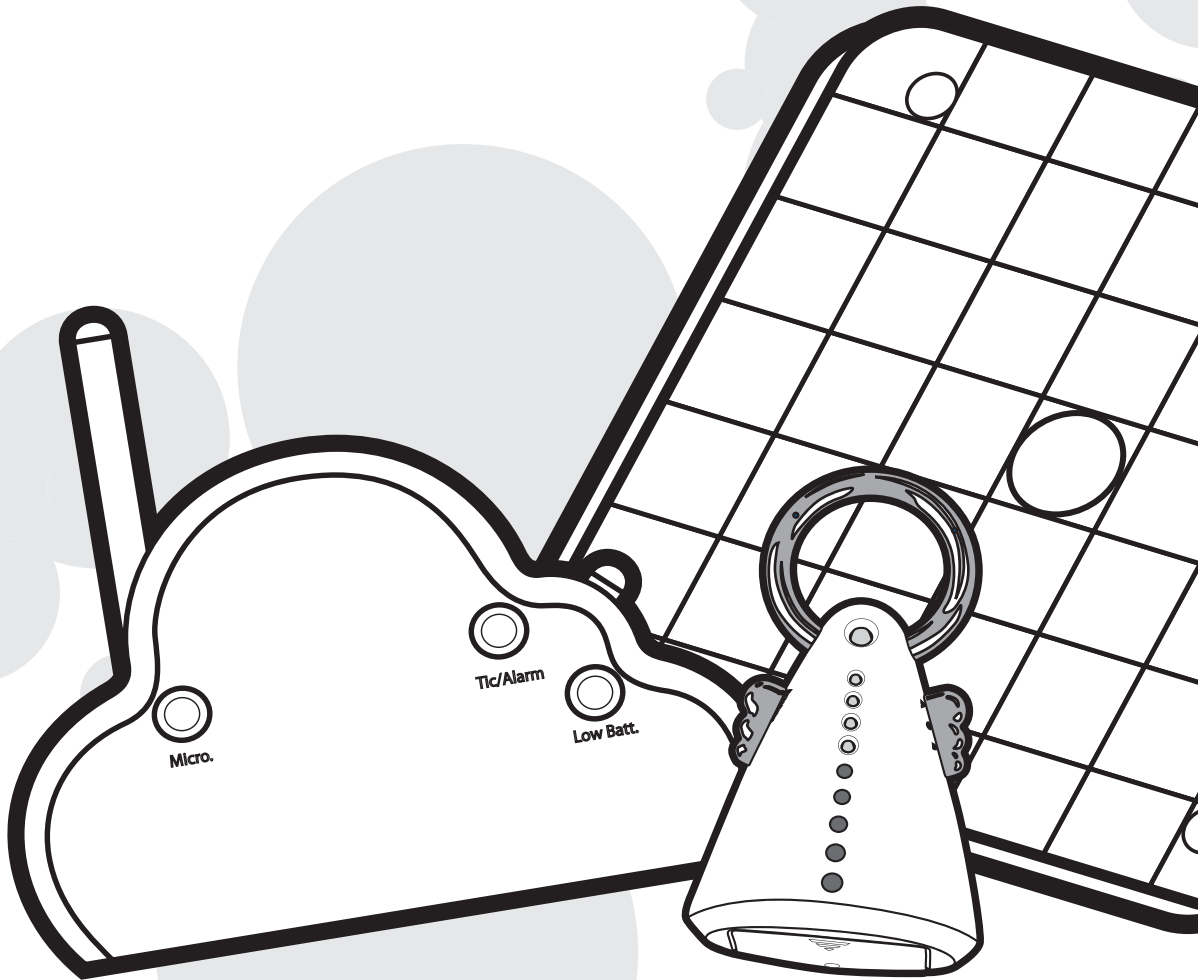
Angelcare®

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AC 201

OWNER'S MANUAL

KEEP INSTRUCTIONS FOR FUTURE REFERENCE.

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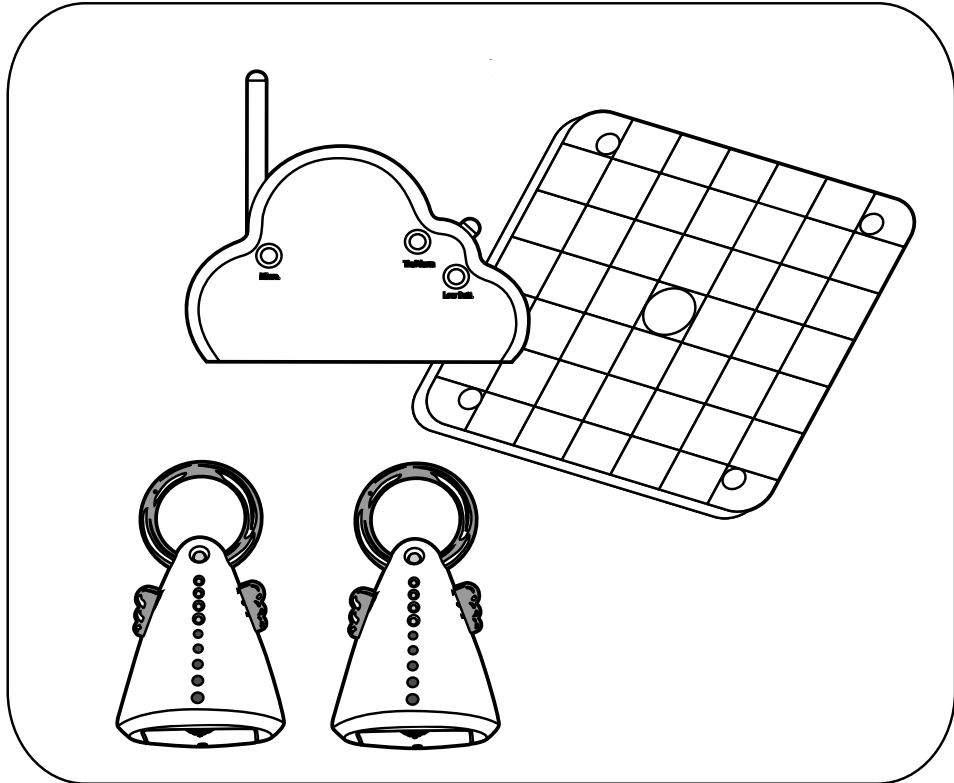
Angelcare®

Model AC201-2P

Your monitor includes 2 parents units. Make sure all units are on the same channel.

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WARNING

Failure to follow these warnings and the assembly instructions could result in serious injury or death.

This product is designed to aid in the monitoring of your child. When using the monitor, you must still provide the supervision necessary for the continued safety of your child. Always be sure that both the transmitter and receiver are working properly and are within range of each other.

- **STRANGULATION HAZARD.** Keep the adapter cords out of the reach of children. Do not use with extension cords.
- **DO NOT** place the Nursery Unit or its cord within your child's reach. Do not put the Nursery Unit inside a crib or playard.
- Use **ONLY** with 110-120 volt AC electrical outlets.
- Protect the AC adapter cords. Place them so they are not walked on or pinched by furniture or other items.
- Allow for proper ventilation when units are in use. Do not cover the Nursery Unit or Parent Unit with any object such as a blanket. Do not place it in a drawer or in any location which would muffle the sound or interfere with the normal flow of air.

The sensor pad is made of plastic and does not permit the air to circulate. Therefore, mildew build up could develop underneath the mattress in the sensor pad area. We suggest rotating the mattress regularly to prevent any mildew.

• ***DO NOT immerse any part of the Nursery Unit or Parent Unit in water. Clean only with a dry cloth.***

• **DO NOT** place unit near water and moisture. Do not use the Nursery Unit outdoors. Do not use the Nursery Unit near possible wet areas, such as a bathtub, shower, wash basin, sink, laundry tub, swimming pool, wet basement, etc.

• Keep Nursery Unit away from heat sources, such as fireplaces, radiators, stoves, and TVs or other appliances. Heat can damage the case or electrical parts.

WARNING: This product is not a toy.

- Do not use the Nursery Unit if:
- The adapter cords or plugs have been damaged.
- The unit has been exposed to liquids.
- Unit has been dropped or is damaged.

• ***DO NOT OPEN THE NURSERY UNIT, PARENT UNIT OR ADAPTERS.***

No user-serviceable parts inside. Risk of electrical shock, fire or death.

• The antenna used for this transmitter must be installed to provide a separation distance of at least 20 cm (7.9") from all persons and must not be co-located or operating in conjunction with any other antenna or transmitter.

DO NOT use these adapters with any other equipment. Use only with the provided adapters.

• When unplugging the AC adapter from the nursery unit, be sure it is switched to the off position. If it isn't, the unit will revert to battery mode if batteries are installed.

- Always place both the Nursery Unit and Parents' Unit on a flat surface in an upright position and allow for proper ventilation.

- Avoid damage to the Sensor Pad cord and to the adapter cords. Route cords so that they are not likely to be walked on or pinched by items placed on or against them.

- Route sensor pad cord only below crib mattress and out of reach of children.

- Use ONLY AAA (LR03) batteries (not included) and/or the AC adapters provided for the Parents' Units.

- Do not mix old and new batteries.

- Do not mix alkaline, standard, (carbon-zinc) or rechargeable (ni-cad, ni-mh, etc.) batteries.

- During long periods of non-use remove all batteries from their compartments and unplug the AC adapters.

- This monitor uses public airwaves to transmit signals. The Parents' Unit may pick up signals or interference from other nursery monitors in your area. Your monitor may even be picked up by other households. To protect your privacy, make sure both units are turned off when not in use. If you experience interference, change the channel on both units.

- This monitor is perfectly safe. The Sensor Pad and its cord are entirely passive. They carry no electrical current and do not radiate any form of energy. The Nursery Unit emits only very low power radio signals through its antenna. These signals do not present a safety hazard.

- When the monitor is in use, do not use a crib mobile or other crib accessories that may produce vibrations. Other sources of vibrations may include fans, washing machines, loud music, etc.

These sources of vibration may interfere with the performance of your Angelcare® monitor and the alarm will not sound if it continues to detect any movement other than your baby's. Be sure all vibration sources are eliminated before using the monitor.

- You may experience false alarms for several reasons. The most common reason is the Nursery Unit has not been turned off after you have removed your baby from the crib. Another reason is the Sensor Pad has not been placed under the mattress properly. Be sure the Sensor Pad is on a flat, firm surface with the turquoise side facing up. The sensitivity level may also need to be adjusted. Refer to the Movement Sensitivity Adjustment section on Pg. 12.

- Always be sure to immediately check on baby whenever an alarm sounds.

- Your monitor can be used in any crib that is stationary and designed to safely hold a baby. If your crib has a spring or slat base you will need to place a piece of Masonite board under the Sensor Pad. This support must be at least 13 inches long on all four sides and 1/4 inch thick. Only use your Angelcare® monitor in a stationary sleeping environment such as a crib where the Sensor Pad can rest on a completely flat, firm surface.

- **CLEANING.** Disconnect all units before cleaning. DO NOT immerse in water. Use a dry cloth to keep monitor free from dust. DO NOT spray with cleaning products or solvents. Wipe sensor pad with a cloth moistened very slightly with an antiseptic or mild detergent.

⚠ WARNING

Modifications not authorized by the manufacturer may void users authority to operate this device.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules.

These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

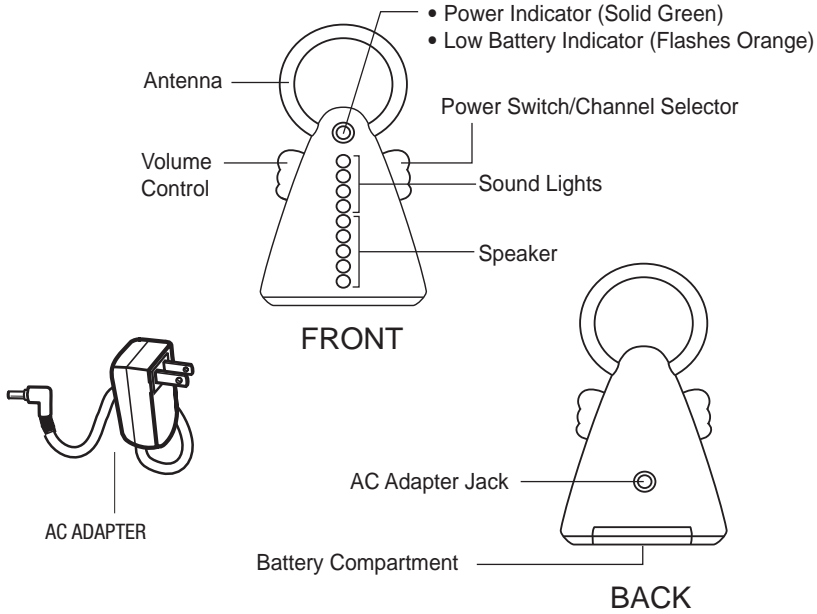
However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.


This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) This device must accept any interference received, including interference that may cause undesired operation.

Features Parent Unit


Nursery Unit

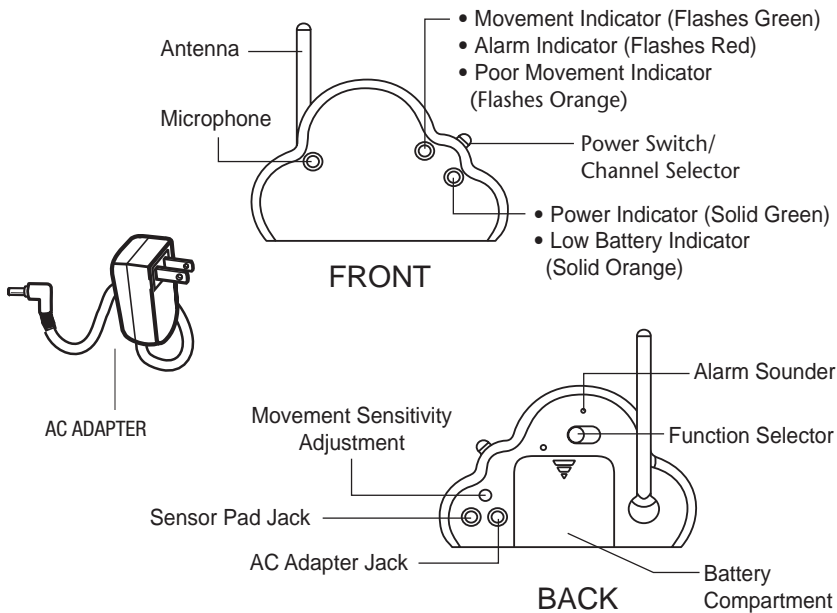
Installing Batteries in Parent Unit • • • • •

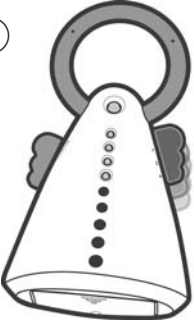①

To insert batteries, remove the battery cover on the bottom of the unit.

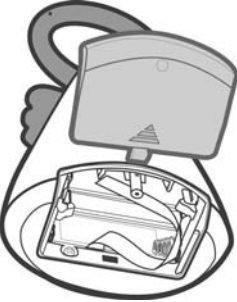
②

The Parents' Unit requires four AAA (LR03) batteries (not included). To insert batteries, remove the battery cover on the back of the unit and insert the batteries matching positive (+) to positive (+) and negative (-) to negative (-) and replace the battery cover.

Replace the batteries when the Low Battery Indicator flashes and turns orange instead of green.

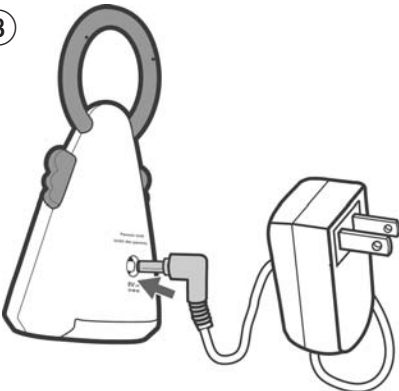③

IMPORTANT: Use only the AC adapters included with your monitor. Do not use these adapters with any other equipment.

Insert the AC adapter into the jack marked "9V (6LR61)" and plug the adapter into an electrical outlet. Unit will run off of the AC adapter unless there is a power outage, in which case the unit will begin using the batteries.

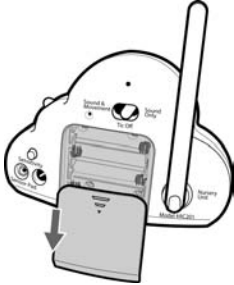
Installing Batteries in Nursery Unit

①


The Nursery Unit requires four AAA (LR03) alkaline or rechargeable batteries (not included).

②


To insert batteries, remove the battery cover on the back of the unit.

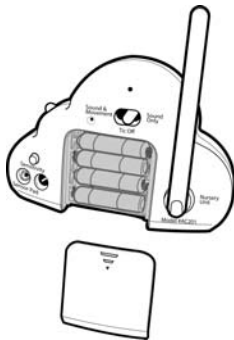③

Insert the batteries matching positive (+) to positive (+) and negative (-) to negative (-). Replace the battery cover.

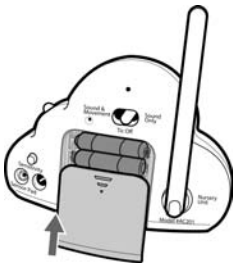
④

Replace the battery cover. Replace the batteries when the Low Battery Indicator turns orange and a rapid ticking is heard. If the batteries are not replaced, the light will eventually turn red and the monitor will stop working.

IMPORTANT: Use only the AC adapters included with your monitor. Do not use these adapters with any other equipment

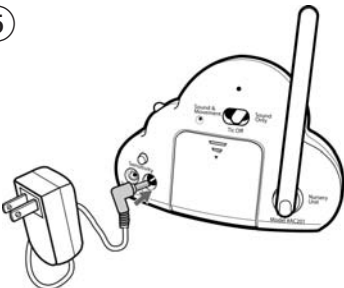⑤

Insert the AC adapter into the jack marked "9V" and plug the adapter into an electrical outlet. Unit will run off of the AC adapter unless there is a power outage, in which case the unit will begin using the batteries.

Setting Up Your Monitor

SENSOR PAD (See figs. 1 and 2)

Place the Sensor Pad under the center of the crib mattress with the printed side facing up. Do not place any loose bedding or linen between the crib mattress and the Sensor Pad. The Sensor Pad must rest on a flat, firm surface. If your crib has a spring or slat base, place a piece of Masonite board under the Sensor Pad. This support must measure at least 13 inches long on all four sides and 1/4 inch thick. If your crib has a plywood base, your mattress support is firm enough to use the monitor properly. Only use your Angelcare® monitor in a stationary sleeping environment such as a crib where the Sensor Pad can rest on a flat, firm surface. Be sure to place the turquoise side of the Sensor Pad facing up.

STRANGULATION HAZARD. Keep the adapter cord and the sensor pad cord out of the reach of children. Because of the serious strangulation risk to infants and toddlers, parents and caregivers **should never place the nursery unit within 3 feet (1 metre) of a crib.** Make sure the Sensor Pad cord passes underneath the mattress entirely. After positioning the Sensor Pad under the mattress, bring the cord to the corner of the crib and secure it to the top and bottom of the leg of the crib, or wind it around the leg (see fig. 1).

Figure 1


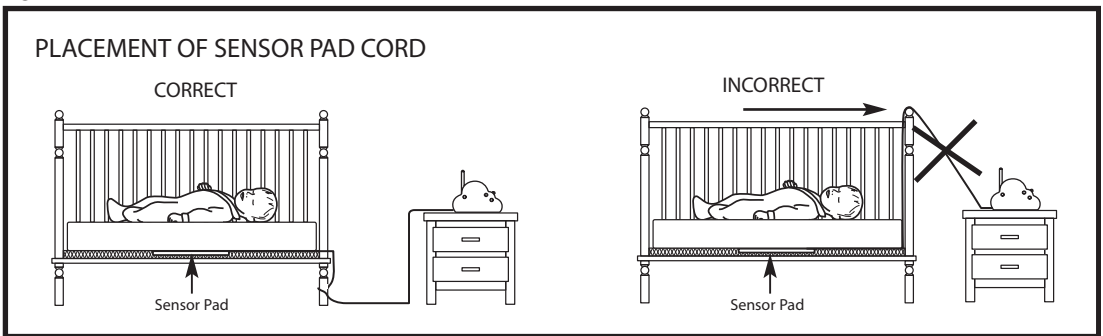
Figure 2

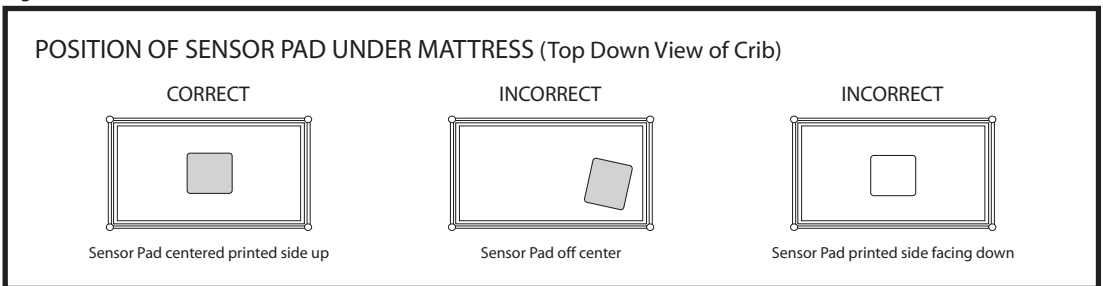Movement Sensitivity Control

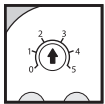
Figure 3

IMPORTANT: Only use the AC adapters included with your monitor. Do not use these adapters with any other equipment. Movement Sensitivity Control. The movement sensitivity control is on the back of the Nursery Unit. The level has been preset to suit most conditions of movement detection (see fig.3). Do not adjust this unless you receive false alarms. Refer to the Movement Sensitivity Adjustment section on Pg. 12 for more information. It is extremely important to follow our instructions for setting the sensitivity level.

Now plug the Sensor Pad cord into the jack on the back of the Nursery Unit marked "Sensor Pad." Turn the Nursery Unit on using the switch on the left. The green Power Indicator will light up.

Activating Units

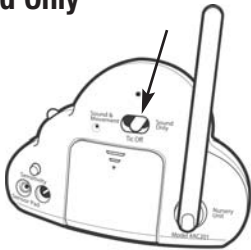
Always test your monitor before using it the first time and then on a periodic basis.
Before testing your monitor, complete the setup process as described above.

On the back of the Nursery Unit select one of the following three functions:

- **Sound & Movement:** You will be able to hear your baby's sounds, see sound lights, and hear a "Tic" sound with every movement your baby makes. If no movement is detected, the "Tic" will stop and after 20 seconds the alarm will sound.
- **Tic Off:** In this mode you will not hear a "Tic" with every movement. You will still hear your baby's sounds and see sound lights, and your baby's movement is still being detected. The default setting is "Tic Off."
- **Sound Only:** The Movement Sensor is disabled but you will still be able to hear your baby's sounds and see sound lights. Your Angelcare® works like a regular sound monitor when set to "Sound Only."


Sound Only

①

Select "Sound Only" as shown.

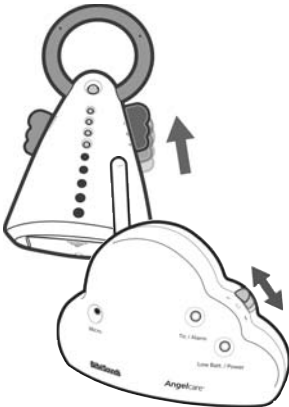②

Turn the Parents' Unit ON by selecting either channel "A" or "B". Make sure it is set to the same channel as the Nursery Unit. The green power indicator light will light up. Choose the channel that gives you the best reception.

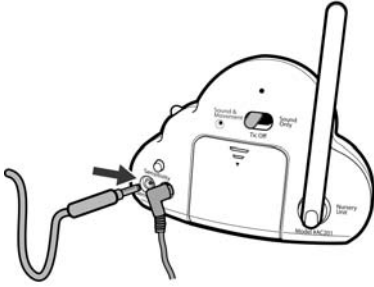
③

Adjust the volume to the desired level as shown.

Sound & Movement • • • • • • • • • •

Note: Make Sure your Sensor Pad is set up before proceeding with this section. If you haven't set up your Sensor Pad please go to Pg.9 for instructions on how to properly set up the Sensor Pad.

- **Sound & Movement:** You will be able to hear your baby's sounds, see sound lights, and hear a "Tic" sound with every movement your baby makes. If no movement is detected, the "Tic" will stop and after 20 seconds the alarm will sound.

Switch the Nursery Unit to the "Sound & Movement" function. Turn the Parents' Unit ON by selecting either channel "A" or "B". Make sure it is set to the same channel as the Nursery Unit. The Movement Indicator Light on the Nursery Unit will flash red, green, and then emit a single "Tic" sound. Gently pat your hand on the mattress. The green Movement Indicator Light will continue to flash with every movement. The Nursery Unit will also tick with every movement.

Remove your hand from the mattress. Due to no movement, the ticking will stop, the Movement Indicator Light will stop flashing, and a pre-alarm beep will sound after 15 seconds. Five seconds later the alarm will sound and the red Alarm Indicator Light will flash continuously.

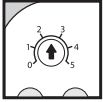
Tic Off

You can also test for movement with the Nursery Unit set to the "Tic Off" position. In this case, you will not hear a constant ticking when you touch the mattress. When you remove your hand from the mattress the Movement Indicator Light will stop flashing, a pre-alarm beep will sound after 15 seconds, and five seconds later the alarm will sound and the red Alarm Indicator Light will flash continuously.

To turn off the alarm, turn the Nursery Unit off or pat your hand lightly on the mattress so the Sensor Pad will redetect movement. The red Alarm Indicator Light on the Nursery Unit will continue to flash for one minute and it will then revert to green.

If the alarm does not sound when you are not touching the mattress, the Sensor Pad may be detecting a continuous vibration from another source from inside and/or outside your baby's room. Common sources of this interference are crib mobiles, fans, washing machines, or even loud music. Therefore, it is very important to make sure all vibration sources are eliminated before using the monitor. Place the crib near a solid supporting wall and away from strong air currents. Do not use crib mobiles or accessories that may cause vibrations. Avoid contact with the crib when the monitor is in use. If you find the alarm will still not sound when you are not touching the mattress or crib, the monitor is on, and you have removed all external sources of vibration, the sensitivity level may need to be decreased. Please refer to the Movement Sensitivity Adjustment section on page 12.

Special Features

Movement Sensitivity Adjustment

The movement sensitivity adjustment dial has been preset to a level which will suit most situations. The highest sensitivity level is 5 (most sensitive) and the lowest sensitivity level is 1 (least sensitive).

You may get false alarms if your baby is in a very deep sleep or has moved to the edge of the crib. If you get false alarms due to these circumstances, you should increase the sensitivity level by using the sensitivity dial on the back of the Nursery Unit.

NOTE: Adjust the sensitivity level if you receive false alarms and you have followed the advice in the Troubleshooting section for false alarms. Make a note of the position of the sensitivity dial before you change it should you wish to return to that setting.

To increase the sensitivity level, start by increasing it by half a level. For example, if the level is currently at 2.5, increase it to 3. Use the monitor as you normally would. If you still experience false alarms, increase it by another half level. If you are still experiencing false alarms when you have reached level 5, please consult our customer service link on our website at www.angelcare-monitor.com

Because the Sensor Pad may detect movement from many sources from both inside and outside your baby's room, be sure all vibration sources are eliminated before changing the movement sensitivity level. Check to see if the Sensor Pad is resting on a flat, firm surface. Place the crib near a solid supporting wall and away from strong air currents. Do not use crib mobiles or accessories that may cause vibrations. Avoid contact with the crib when monitor is in use.

To make sure you have avoided all external sources of vibration, take your baby out of the crib while the monitor is on. If the alarm does not sound after you take your baby out of the crib, the sensitivity level may need to be decreased. To decrease the sensitivity level, adjust by half a level by going from 2.5 to 2. If you are still experiencing this problem when the level is set to 1, please consult our customer service link on our website at www.angelcare-monitor.com

Sound Lights

The Parents' Unit features Sound Lights so you can "see" your baby's sounds if background noise prevents you from hearing your baby. As the sounds get louder, more lights illuminate.

Optional "Tic" Feature (Page. 10)

When the "Tic" feature is selected, the Parents' Unit will continuously tick as your baby's movement is detected by the Sensor Pad. If no movement is detected, the ticking will stop. If you do not want to hear the ticking, slide the switch to the "Tic Off" position.

Movement Indicators

The Movement Indicator Light is located only on the Nursery Unit. When the Nursery Unit is set to "Sound & Movement" or "Tic Off," the Movement Indicator Light will flash green with your baby's every movement. The flashing green light will let you know that movement is being detected. This will allow you to check on your sleeping baby in the dark without disturbing.

Tips For Using Your Monitor • • • • •

- When the monitor is in use, do not use a crib mobile or other crib accessories that may produce vibrations. Other sources of vibrations may include fans, washing machines, loud music, etc. These sources of vibration may interfere with the performance of your Angelcare® monitor and the alarm will not sound if it continues to detect any movement. Be sure all vibration sources are eliminated before using the monitor.
- There are several reasons you may experience false alarms. The most common reason is the Nursery Unit has not been turned off after you have removed your baby from the crib. Another reason is the Sensor Pad has not been placed under the mattress properly. Be sure the Sensor Pad is on a flat, hard surface with the printed side facing up. The sensitivity level may also need to be adjusted. Refer to the Movement Sensitivity Adjustment section on page 12.
- Your monitor can be used in any crib that is stationary and designed to safely hold a baby. If your crib has a spring or slat base you will need to place a piece of Masonite board under the pad. This support must be at least 13 inches long on all four sides and 1/4 inch thick. Only use your Angelcare® monitor in a stationary sleeping environment such as a crib where the Sensor Pad can rest on a hard, flat surface.
- Please note the difference between the Low Battery Indicator and the Alarm Indicator on the Nursery Unit. The Low Battery Indicator is a rapid ticking sound and the Low Battery Indicator Light turns orange. The Alarm Indicator is a loud beeping sound and the red Alarm Indicator Light flashes.
- When unplugging the AC adapter from each unit, be sure the unit is switched to the off position. If it isn't, the unit will revert to battery mode if batteries are installed.
- You can use rechargeable batteries, however, a rechargeable battery cannot be recharged by plugging in the AC adapter. Remove batteries from the unit to recharge.
- The range of this monitor is approximately 300 feet. However, the range will vary depending on the surroundings. Test the range before use. NOTE: This monitor is not equipped with an out-of-range indicator. No sound will be received if the Nursery Unit and Parents' Unit are too far apart.

Testing Your Monitor

Always test your monitor before using it the first time and then on a periodic basis. Before testing your monitor, complete the set-up process as described above.

1. Testing for Sound. Turn the Nursery Unit and Parents' Unit on, making sure they are both set to the same channel. On the Nursery Unit, switch to the "Sound Only" function. Have someone else speak into the Nursery Unit or place a radio near the Nursery Unit while you take the Parents' Unit into another room. You should hear the sounds being transmitted from the Nursery Unit. If there is no sound, adjust your volume control and/or refer to the Troubleshooting section on page 15. If you experience interference or static, switch both units to the other channel.

2. Testing for Movement. Switch the Nursery Unit to the "Sound & Movement" function. The Movement Indicator Light on the Nursery Unit will flash red, green, and then emit a single "Tic" sound. Gently pat your hand on the mattress. The green Movement Indicator Light will continue to flash with every movement. The Nursery Unit will also tick with every movement. Remove your hand from the mattress. Due to no movement, the ticking will stop, the Movement Indicator Light will stop flashing, and a pre-alarm beep will sound after 15 seconds. Five seconds later the alarm will sound and the red Alarm Indicator Light will flash continuously.

You can also test for movement with the Nursery Unit set to the "Tic Off" position. In this case, you will not hear a constant ticking when you touch the mattress. When you remove your hand from the mattress the Movement Indicator Light will stop flashing, a pre-alarm beep will sound after 15 seconds, and five seconds later the alarm will sound and the red Alarm Indicator Light will flash continuously.

To turn off the alarm, turn the Nursery Unit off or pat your hand lightly on the mattress so the Sensor Pad will redetect movement. The red Alarm Indicator Light on the Nursery Unit will continue to flash for one minute and it will then revert to green.

If the alarm does not sound when you are not touching the mattress, the Sensor Pad may be detecting a continuous vibration from another source from inside and/or outside your baby's room. Common sources of this interference are crib mobiles, fans, washing machines, or even loud music. Therefore, it is very important to make sure all vibration sources are eliminated before using the monitor. Place the crib near a solid supporting wall and away from strong air currents. Do not use crib mobiles or accessories that may cause vibrations. Avoid contact with the crib when the monitor is in use.

If you find the alarm will still not sound when you are not touching the mattress or crib, the monitor is on, and you have removed all external sources of vibration, the sensitivity level may need to be decreased. Please refer to the Movement Sensitivity Adjustment section on page 12.

3. Testing for Battery Power. Verify that the units are operating on battery power by installing four AAA batteries in each unit and unplugging the AC adapters. The green Power Indicator should light up on all units.

Care and Maintenance

Disconnect all units and remove all batteries before cleaning. Clean the exterior surface of the units with a soft dry or slightly damp cloth. Do not spray with cleaning products or solvents. Do not submerge in water or any other liquid.

During long periods of non-use, remove all batteries from their compartments and unplug the AC adapters.

Troubleshooting

Problem	Possible Cause	Solution
False alarms	Baby was removed from crib and the Nursery Unit was not switched off.	Turn the Nursery Unit off.
	The printed side of the Sensor Pad is not facing up.	Make sure the printed side is facing up.
	The Sensor Pad is not detecting your baby's movements because the sensitivity level is too low.	The sensitivity level may need to be increased. Refer to the Movement Sensitivity Adjustment section on Pg. 12.
	Sensor Pad is not in full contact with the crib mattress or is not resting on a flat, firm surface.	Make sure there is no thick bedding between the mattress and the Sensor Pad. The Sensor Pad must rest on a firm surface. If the crib has a spring or slat base, place a piece of Masonite board measuring at least 13" on all four sides and 1/4" (0.64cm) thick.
	Baby has moved far away from the Sensor Pad.	Make sure there is no thick bedding between the mattress and the Sensor Pad. The Sensor Pad must rest on a firm surface. If the crib has a spring or slat base, place a piece of Masonite board measuring at least 13" (33cm) on all four sides and 1/4" (0.64cm) thick. Refer to the Movement Sensitivity Adjustment section on Pg. 12.

Problem	Possible Cause	Solution
False alarms	Sensor Pad cord is not connected properly to the Nursery Unit.	Unplug and reconnect the cord.
Alarm will not sound	Nursery Unit is detecting movement when crib is being touched.	Avoid contact with the crib when monitor is on.
	Nursery Unit is detecting movement from outside the crib such as motorized devices, strong drafts, etc.	Place crib near a solid supporting wall to avoid Sensor Pad from detecting movement from outside the crib. Turn off crib mobile, fans, loud music or anything else that may cause vibrations.
	Sensor Pad sensitivity level is set too high.	You may need to decrease the sensitivity. Refer to the Movement Sensitivity Adjustment section on Pg. 12.
Movement Indicator Light on Nursery Unit is flashing orange (monitor is barely detecting movement).	Sensor Pad is not in full contact with crib mattress, or the mattress is not resting on a hard surface.	Make sure there is no loose bedding between mattress and the Sensor Pad. The Sensor Pad must rest on a hard surface. If the crib has a spring or slat base, place a piece of Masonite measuring at least 13" (33cm) on all four sides and 1/4" (0.64cm) thick on top of the base.
	Baby is in a very deep sleep or has moved away from Sensor Pad to the edge of the crib.	Make sure Sensor Pad is positioned correctly in crib. The sensitivity level may need to be adjusted. Refer to the Movement Sensitivity Adjustment section on page 12.
No sound is being transmitted.	Parents' and Nursery Units are too far apart.	Bring Parents' Unit closer to Nursery Unit.
	Batteries are low or improperly installed.	Check or replace batteries.
	AC adapters are not connected properly or electrical outlet is out of order. no está energizado.	Check connections or change to another electrical outlet.
	One of the units is not on.	Make sure both units are on.
	Units are set to different channels.	Make sure both units are on the same channel, either both on A or B.
	Volume level is too low.	Increase volume level.

Problem	Possible Cause	Solution
Power Indicator (green light) does not light up.	Batteries are low or improperly installed.	Check or replace batteries.
	AC adapters are not connected properly or electrical outlet is out of order.	Check connections or change to another electrical outlet.
	One of the units is not on.	Make sure both units are on the same channel, either both on A or B.
There is very weak reception.	Parents' and Nursery Units are too far apart.	Bring Parents' Unit closer to Nursery Unit.
	One or both units are placed near a large metal mass.	Change the position of one or both units.
	One or both units are not in the upright position.	Place units on flat surface away from baby's reach.
	Batteries are low.	Check or replace batteries.
There is static, distortion, interference, etc.	Parents' and Nursery Units are too far apart.	Bring Parents' Unit closer to Nursery Unit.
	Units are not set to the same channel.	Make sure both units are on the same channel, either both on A or B.
	One of the units is not on.	Make sure both units are on.
	The Parents' Unit is located near motorized devices, fluorescent lights, television, etc.	Rotate Parents' Unit or move it away from source of interference.
There is feedback (squealing noises).	Units are too close to each other.	Move units farther apart (at least 10 feet apart).
	The volume is too high.	Decrease volume on Parents' Unit.

IMPORTANT! This Angelcare® monitor is a personal care product. Please do not return this product to the store. If you have questions or need help, please call Angelcare before returning this product.

Replacement Parts
Warranty Information (USA & Canada only)
www.angelcare-monitor.com
angelcare@cdecinc.ca
1-877-281-8827

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*The ANGELCARE® trademark is a registered trade mark of the Angelcare Development Inc.
La marque ANGELCARE® est une marque enregistrée propriété de Les Développements Angelcare Inc.
La marca ANGELCARE® es una marca registrada de Angelcare Development Inc.*